BodyMetrix.COM.TR

ULTRASON TEKNOLOJİ

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ALANYA Mahmutlar 07450 – Willi Ehrlich – PREVENTION21.COM

PREVENTION21[®].SHOP

PREVENTION21.COM: The BodyMetrix System offers the same professional-grade ultrasound technology used by elite athletes and trainers. This high tech, cutting edge device allows you to quickly, easily and accurately make assessments. The BodyMetrix System does all this without embarrassing or painful pinching. Assessments take only minutes and are not affected by hydration, exercise level or caffeine intake. Since the BodyMetrix device can be plugged into your Windows or Mac desktop or laptop and Windows Pro tablets, you can make assessments anywhere.

Accurately Measure Body Composition

Body composition is calculated by measuring subcutaneous fat thickness at multiple sites. The number of sites depend on the formula and can range from an approximate single bicep measurement to as many as 9 sites. BodyMetrix™ Ultrasound measurements are not sensitive to hydration or exercise. Screenshots below show sample measurements using a single bicep formula on an overweight individual, and a more accurate 7 site formula on an elite athlete.

BodyViewProFit software

Beyond body composition BodyViewProFit software can be used to track individual point fat thickness and circumference, waist to hip ratio and fitness metrics. The two fitness lists shown below can be easily modified to only include what you need.

Accurately Measure Fat and Muscle directly, Seeing is Believing

Beyond point measurements the BodyMetrix can be used to scan anatomical sites to directly see fat and muscle layers as can be see below in a thigh scan. Scans offer an excellent way to directly monitor the benefits of diet and exercise. They are also a terrific way to identify skinny fat individuals who often aren't convinced by numbers.

Abdominal subcutaneous fat is sensitive to diet and exercise.

Abdominal scans below show double layer for high %BF (29%) individuals compared to a single layer for low %BF (3%) elite athlete. These abdominal scans start near the belly button and extend to within a few inches of the hip bone.

Muscle Change

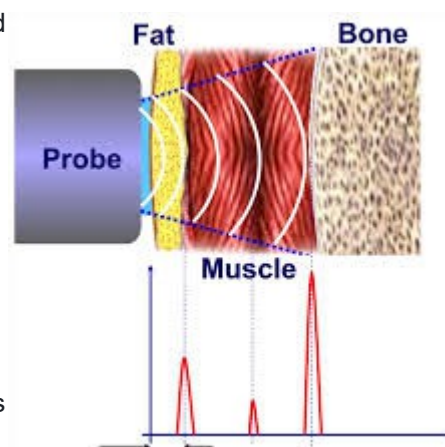
Capturing scans over time allows you to directly see change in fat and muscle.

Accurately Measure Body Composition

Body composition is calculated by measuring subcutaneous fat thickness at multiple sites.

Muscle Imbalance

The scans below taken of the left and right quadricep from the midpoint to near the patella were recorded on the same day. The scans show clear differences. The yellow and red arrows represent the same thickness in each image, showing significant imbalance. The ratio of the Rectus Femoris thickness to the Vastus intermedius is 1.3 for the left leg and 0.8 for the right leg.

Accurately Measure Body Composition

Body composition is calculated by measuring subcutaneous fat thickness at multiple sites.

Ultrasound and Body Composition

Ultrasound was first used to measure body composition in the 80's.

Measuring fat thickness at multiple sites has proven to be an accurate technique to measure body composition (%BF). The benefit of ultrasound is that you can also measure muscle thickness directly. Ultimately BodyMetrix™ body composition measurements will use both fat and muscle thickness to increase accuracy.

Below are summaries of several studies which have compared the BodyMetrix to DEXA, BodPod, Hydrostatic, and Bio-impedance. The studies show good agreement with these more expensive and non-portable systems.

VIEW: www.EHRLICH.PL and my „**Unbreakable-Immune-System.COM**“ over 25 years without a doctor!

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